Reusable Adaptor & Extension Cable for Temperature Probe

English

Indications: FMT temperature adaptor and extension cables are used to transmit electrical signals received from the reusable or disposable medical temperature probe applied to the patient to the thermometer or patient monitor for continuous monitoring of body temperature. Temperature adaptor and extension cables are intended to be used by qualified medical personnel and can be used in infant, pediatric and adult patients, depending on the selection of the appropriate size disposable or reusable temperature probe in hospitals and hospital-type facilities.

Contraindications: No known contraindications to the use of the product.

Adverse Reactions: No known adverse reactions to the use of the product.

Clinical benefits: Enables temperature monitoring with patient monitoring equipment.

Product Description: FMT temperature adaptor & extension cable have different probe-side connectors, device-side connectors, and connector wiring configurations for different medical thermometer or patient monitor manufacturers. Therefore, the same appearance does not guarantee full compatibility with the medical thermometer or patient monitor. Always consider the information on the label on the product and/or its primary packaging.

FMT400/AEC (-XX) adaptor & extension cable with Molex type socket on the probe side (for disposable temperature probes), 2.4 m
FMT400/AEC (-PXX) adaptor & extension cable with 3.5 mm (1/8") mono plug on the probe side (for disposable temperature probes), 2.4 m
FMT400/AEC (-RXX) adaptor & extension cable with RJ11 jack on the probe side (for disposable temperature probes), 2.4 m
FMT400/AEC (-EXX) adaptor & extension cable with 6.35 mm (1/4") mono jack on the probe side (for reusable temperature probes), 2.4 m
(XX: Empty for standard 6.35 mm angled mono plug, -HP for PHILIPS 2 pin, -S for DRAGER/SIEMENS 7 pin, -SW for S&W, -GE for 11 pin GE, -MN for 2 pin MINDRAY, -BL for 2 pin BIOLIGHT, and -MMM for BELMONT 6.35 mm (1/4") angled stereo connector on the device side)
FMT400/AEC/Z (-X) adaptor & extension cable with 6.35 mm (1/4") straight mono plug on the device side, 2.4 m
(X: Empty for Molex type socket, -E for 6.35 mm (1/4") mono jack, -P for 3.5 mm (1/8") mono jack, -R for RJ11 jack on the probe side connector)
FMT400/AEC-THT adaptor & extension cable with 2 pin REDEL connector on the device side, Molex type socket on the probe side, 2.4 m
FMT400/AEC-GM/Z adaptor & extension cable with 6.35 mm (1/4") straight stereo plug on the device side, Molex type socket on the probe side, 2.4 m
FMT400/AEC-EGES adaptor & extension cable with 11 pin GE type connector on the device side, 6.35 mm (1/4") mono jack on the probe side, 0.5 m
FMT400/AEC-EGE2 dual adaptor with two 6.35 mm (1/4") mono jacks on the probe side, 11 pin GE type connector on the device side, 0.45 m
FMT400/AEC-EGE3 dual adaptor with two 6.35 mm (1/4") mono jacks on the probe side, 11 pin GE type connector on the device side, 0.45 m
FMT400/AEC-SPL dual adaptor with two 6.35 mm (1/4") mono jacks on the probe side, 10 pin SPACELABS type connector on the device side, 0.45 m
FMT400/AEC-NKN dual adaptor with two 6.35 mm (1/4") mono jacks on the probe side, 14 pin NIHON KOHDEN type connector on the device side, 0.45 m

Directions for Use:

- 1- Before use, check that the product is intact and clean.
- 2- Connect the medical temperature probe to the probe side connector of the temperature adaptor & extension cable.
- 3- Connect the temperature adaptor & extension cable connector to the socket of the compatible medical thermometer or patient monitor and check it for correct operation.
- 4- Apply the medical temperature probe to the patient as indicated in the medical temperature probe instructions for use and wait for the probe temperature to stabilize.
- 5- Please refer to the medical thermometer instructions for use for detailed information on temperature monitoring.

Warnings:

- 1- All temperature adaptor & extension cables are designed for use with specific medical temperature probes and medical thermometers. Verify the compatibility of the thermometer, medical temperature probe, and all functions properly before use, otherwise degraded performance may result.
- 2- Before each use, visually inspect the temperature adaptor & extension cables. The temperature adaptor & extension cables should be free of visible defects and damage. If there are breaks, scratches, or cracks on the cable jacket or plastics insulation of the connector, or there are visibly bent, damaged pins of the connector discontinue using the cable. Never use a damaged adaptor & extension cables or one with exposed electrical circuitry. Even if there is no exposed electrical circuit, the scratches or cracks in the cable jacket or plastic insulation may cause the accumulation of pathogens and reduce the effectiveness of cleaning and disinfection of the product. Dispose of damaged adaptor & extension cables according to hospital disposal practices, local environmental and waste disposal laws, and regulations.
- 3- Do not use adaptor & extension cables during Magnetic Resonance Imaging (MRI) scanning or in an MRI environment as it may result in physical harm.
- 4- Keep the adaptor & extension cables connectors as far as possible from electromagnetic sources such as the power transformer of an electromedical device. Also, avoid placing the adaptor & extension cables alongside the power cord of an electromedical device. Strong electromagnetic fields may adversely interfere with the function of an adaptor & extension cable.
- 5- As with all medical equipment, carefully route cables to reduce the possibility of patient entanglement or strangulation.
- 6- To prevent shock hazards do not connect adaptor & extension cables to any power outlet.
- 7- To avoid a shock hazard and interface from nearby electrical equipment, keep adaptor & extension cables away from grounded metal and other electrical equipment.
- 8- When connecting and disconnecting the adaptor & extension cables pull on the connector itself, not on the cable.
- 9- Do not modify the adaptor & extension cables in any way. Modifications may affect patient safety, performance, and accuracy and void the warranty
- 10- Mishandling of the adaptor & extension cables could result in damage to internal wires or loss of electrical isolation. Don't tie the adaptor & extension cables and in knots or bunch tightly. Don't wrap adaptor & extension cables around equipment to avoid damaging internal wires. Don't leave the adaptor & extension cables dangling from the instrument. Keep adaptor & extension cables away from moving parts that might pinch or crush them. When not in use, loosely coil the adaptor & extension cables to a bend diameter of at least 10 cm and store it at room temperature.
- 11- Mating connectors with force without proper alignment may damage the connectors and disrupt the electrical continuity.
- 12- For further information and warnings read the instruction manual of the medical thermometer or patient monitor before using the adaptor & extension cables.

Manual Cleaning and Disinfection:

Manual Cleaning: The temperature adaptor & extension cables must be cleaned and disinfected immediately after each use. Remove heavy debris from the temperature adaptor & extension cable by wiping with an appropriate, moist lint-free cloth, sponge, or gauze pad. Prepare the detergent solution with reverse osmosis or distilled water according to the manufacturer's instructions. Saturate a synthetic felt or sponge pad with the cleaning solution. Wipe all surfaces of the temperature adaptor & extension cable until all visible contamination is removed. Wash the pad in the cleaning solution and repeat wiping the cable. Use a plastic cleaning brush for adhered particles or dried secretions. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 2 minutes. If visible contamination is still present on the cable at the end of the cleaning repeat the cleaning process. Moist a clean cloth, sponge, or gauze pad with tap water or reverse osmosis and thoroughly wipe the temperature adaptor & extension cable to remove any debris and all traces of the cleaning solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

Manual Disinfection: Prepare the disinfectant solution according to the manufacturer's instructions. Only disinfect clean products that have dried slightly. Saturate a synthetic felt or sponge pad with the disinfectant solution. Wipe all surfaces of the temperature adaptor & extension cable. Wash the

pad in the disinfectant solution and repeat wiping the cable. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 6 minutes. Moist a clean cloth, sponge, or gauze pad with reverse osmosis or distilled water and thoroughly wipe the temperature adaptor & extension cable to remove any residual debris and all traces of disinfectant solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

Temperature adaptor & extension cable, especially metallic parts of the connector should never be immersed in cleaning, disinfecting, or rinsing solutions, or placed in an ultrasonic cleaning device. Soaking will reduce the life of the temperature adaptor & extension cable by oxidation of metal parts and hardening of the cable jacket. Avoid using excessive force when cleaning or disinfecting by hand which may damage the internal wires of the cable and reduces the product life. Please do not clean with hard or frictional materials, abrasive cleaners, or solvents. Cleaning agents containing phenol or chloride are not suitable. The temperature adaptor & extension cable should be completely dry before use, do not use wet temperature adaptor & extension cables. These instructions were validated using the below detergents and disinfectants.

Cleaning agent (manual cleaning): Enzol® / Cidezime®, enzymatic detergent, Johnson&Johnson (add 8ml enzymatic detergent per 1 liter distilled water, for cable with dried-on organic matter use 16 ml enzymatic detergent per 1 liter distilled water/ warm water)

Disinfectant (manual disinfection): Cidex OPA®, 0.55% ortho-phthaladehyde solution, Johnson&Johnson (at a minimum temperature of 20°C/68°F).

Reverse osmosis/ distilled water.

Caution: During the cleaning and disinfection of the temperature adaptor & extension cables disposable gloves, protective eyewear, and filtration mask should be used to reduce the risk of transmission of infectious agents by splashing or the creation of aerosols.

Warning: FMT temperature adaptor & extension cables are sold NON-STERILE. Clean and disinfect the cables before first and every use.

Warning: Before cleaning or disinfecting the cable, disconnect it from the medical thermometer.

Warning: Temperature adaptor & extension cables should be cleaned of overburden prior to disinfection to improve the effectiveness.

Warning: Do not sterilize temperature adaptor & extension cables by autoclave, radiation, or steam.

Warning: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents. Do not use bleach on electrical contacts or connectors.

Environmental Requirements:

Operating Temperature : 5°C - 45°C (41°F - 113°F)

Storage Temperature : 0°C - 50°C (32°F - 122°F)

Relative Humidity : 20 - 80% (not condensing)

During the storage, the products should be protected from sunlight. It is recommended to store the product in its original packages until the first use.

Used cables should be loosely coiled in 10 to 15 cm diameter (4 to 6 inches) and must be stored individually in a protective container or packaging.

Compatibility: In order to ensure compatibility, the temperature adaptor & extension cables should only be used with the equipment for which they have been designed and are specified in the label on the product's primary packaging. For product models and compatible devices, please refer to the catalog at metkomedical.com/PDF/04MedicalTemperatureProbes.pdf

Safety:

Degree of protection from electric shocks: type BF

Degree of protection against the ingress of water: IPX1

The FMT reusable temperature adaptor & extension cables are sold NON-STERILE.

The FMT reusable temperature adaptor & extension cables are not made with natural rubber latex.

The FMT reusable temperature adaptor & extension cables have not been evaluated for safety in the MR environment. Scanning a patient who has this device may result in patient injury.

Refer to the separate leaflet "Description of Symbols" located within the product package for the explanation of symbols.

Reporting of serious incidents: Any serious incident related to the use of this product should be reported to both the manufacturer and the health authority/competent authority where the product is installed. Either contact your local representative or report to: metko@metkomedical.com

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Warranty: FMT temperature adaptor & extension cables are under twelve (12) months warranty against material and workmanship defects from the date of original purchase. In warranty period, METKO will be responsible for repairing the cable or change the cable free of charge if the defect is proven. This warranty does not extend to any product that has been subject to misuse, wrong disinfection, neglect or accident; or that has been damaged by causes external to the product, or that has been used in violation of the operating instructions supplied with the product. The product life is 3 years of the production date.

FMT® is a registered trademark of Metko A.Ş.

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