Indications: Patient monitoring ECG cables are used to convey electrocardiographic signals of a patient from the ECG electrodes placed on the patient's skin to a patient monitor, a device that processes these signals. ECG cables are intended to be used by qualified medical personnel on infants, children, and adults in hospitals and hospital-type facilities.

Contraindications: No known contraindications to the use of the product.

Adverse Reactions: No allergic reactions are observed at places where ECG cables are in contact with the intact skin of a patient. However, such reactions cannot be completely excluded in people particularly susceptible to allergies; it should be noted that no such events have been reported to the manufacturer. Incorrect cleaning or disinfection, or a failure to perform them, as well as failure to observe hygiene procedures, may cause the transfer of pathogens and infections.

Clinical Benefits: Enables ECG monitoring with patient monitoring equipment.

Product Description: FMT ECG cables have different device-side connectors and connector wiring configurations for different patient monitor manufacturers. Therefore, the same appearance does not guarantee full compatibility with the patient monitor. Always consider the information on the product's label and/or primary packaging. Snap or grabber-type patient terminations are available.

E200-Xxxx/YZ, ECG cable with 3, 4, or 5 fixed patient leadwires.

E201-Xxxx/YZ, ECG cable with 3 or 5 1.6 mm detachable Safety DIN patient leadwires.

E202-Xxxx/YZ, ECG cable with 3 or 5 detachable SIEMENS Type patient leadwires.

E204-Xxxx/YZ, ECG cable with 3 or 5 detachable GE Type patient leadwires.

E205-Xxxx/YZ, ECG cable with 3 or 5 detachable PHILIPS - HP Type patient leadwires.

E206-Xxxx/YZ, ECG cable with 3 or 5 detachable SPACELABS Type patient leadwires.

E207-Xxxx/YZ, ECG cable with 3 or 6 detachable NIHON KOHDEN Type patient leadwires.

E208-Xxxx/YZ, ECG cable with 3 or 5 detachable PHILIPS - HP Type patient leadwires.


E209-Xxxx/YZ, ECG cable with 3, 4, 5 or 6 fixed patient leadwires.

(X shows the number of leadwires, 3, 4, 5, or 6. xxx shows the compatible brand. Y shows patient termination type S=Snap, G=Grabber. Z shows the color code I=IEC, A=AHA/AAMI)

Directions for Use:

- 1- Before use, check that the product is intact and clean.
- 2- Connect the ECG cable connector to the socket of the compatible patient monitor and check it for correct operation.
- 3- Before applying the electrodes, ensure the patient's skin is clean, dry, and properly prepared. Always apply the electrodes to intact skin.
- 4- Attach leadwire terminations to the electrodes. Ensure the leadwires do not pass twist stress onto electrodes.
- 5- Apply the electrodes firmly on prepared sites using the chart given below.
- 6- Ensure the leads conform to the body contours and no strain is placed on the electrodes.
- 7- Verify that the ECG signal appears on the patient monitor screen. For detailed information about ECG monitoring, please refer to the patient monitor instructions for use.

IEC Color code		AHA/AAMI Color code		Location
R	Red	RA	White	Right mid-clavicular line, just below the clavicle.
L	Yellow	LA	Black	Left mid-clavicular line, just below the clavicle.
F	Green	LL	Red	7th intercostal space in line with or lateral to the midpoint of the left clavicle.
N	Black	RL	Green	7th intercostal space in line with or lateral to the midpoint of the right clavicle.
C	White	V	Brown	Intercostal space equivalent to the C (V)

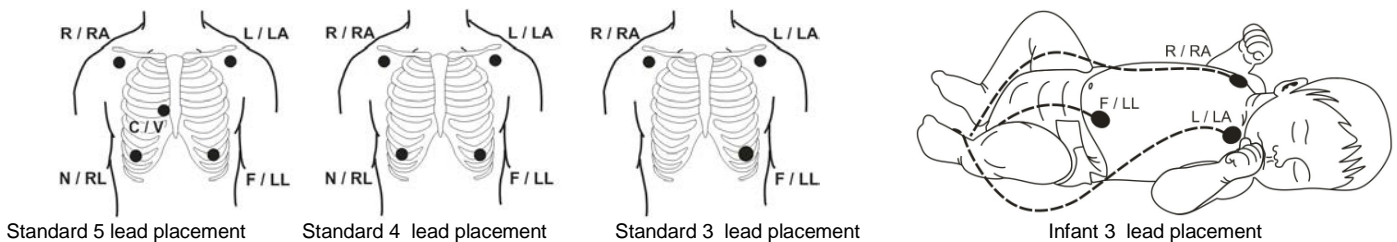Warnings:

- 1- All ECG cables are designed for use with a specific patient monitor. Before use, verify the compatibility of the patient monitor and ECG cable and that all functions work properly. Otherwise, degraded performance may result.
- 2- Before each use, visually inspect the ECG cable. The ECG cable should be free of visible defects and damage. If there are breaks, scratches, or cracks on the cable jacket or plastic insulation of the connector or patient terminations, or there are visibly bent, damaged pins of the connector or patient terminal, discontinue using the cable. Never use a damaged cable or one with exposed electrical circuitry. Even if there is no exposed electrical circuit, the scratches or cracks in the cable jacket or plastic insulation may cause the accumulation of pathogens and reduce the effectiveness of cleaning and disinfection of the product. Dispose of damaged cables according to hospital disposal practices, and local environmental and waste disposal laws and regulations.
- 3- Do not use the ECG cable during Magnetic Resonance Imaging (MRI) scanning or in an MRI environment, as it may cause physical harm.
- 4- Keep the ECG cable connectors as far as possible from electromagnetic sources such as an electromedical device's power transformer. Also, avoid placing the ECG cable alongside the device's power cord. Strong electromagnetic fields may adversely interfere with the function of an ECG cable.
- 5- As with all medical equipment, carefully route cables to reduce the possibility of patient entanglement or strangulation.
- 6- To prevent shock hazards, do not connect the ECG cable to any power outlet.
- 7- To avoid a shock hazard and interface with nearby electrical equipment, keep electrodes and patient cables away from grounded metal and other electrical equipment.
- 8- When connecting and disconnecting the ECG cable and the patient terminations pull on the connector or plug itself, not on the cable.
- 9- Do not modify the ECG cable in any way. Modifications may affect patient safety, performance, and accuracy and void the warranty.
- 10- Mishandling the ECG cables could damage internal wires or cause loss of electrical isolation. Do not tie the ECG cable and leadwires in knots or bunch tightly. Do not wrap ECG cables around equipment to avoid damaging internal wires. Do not leave the ECG cable dangling from the instrument. Keep the leadwires away from moving parts that might pinch or crush them. When not in use, loosely coil the ECG cable to a bend diameter of at least 10 cm and store it at room temperature.
- 11- Dirty, compromised or unprepared skin may cause incorrect ECG readings. Using previously opened or out-of-date electrodes may degrade the ECG signal quality. ECG electrodes should be removed from their sealed package immediately prior to use.
- 12- If all the leadwires are not connected correctly with the patient, that may cause reading errors.
- 13- Mating connectors with force without proper alignment may damage the connectors and disrupt the electrical continuity.
- 14- Before using the ECG cable, read the instruction manual of the ECG device for further information and warnings.

Manual Cleaning and Disinfection:

Manual Cleaning: The ECG cables must be cleaned and disinfected immediately after each use. Remove heavy debris from the ECG cable by wiping it with an appropriate, moist, lint-free cloth, sponge, or gauze pad. Prepare the detergent solution with reverse osmosis or distilled water according to the manufacturer's instructions. Saturate a synthetic felt or sponge pad with the cleaning solution. Wipe all surfaces of the ECG cable until all visible contamination is removed. Wash the pad in the cleaning solution and repeat wiping the cable. Use a plastic cleaning brush for adhered particles or dried secretions. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 2 minutes. If visible contamination is still present on the cable at the end of the cleaning process, repeat the cleaning process. Moist a clean cloth, sponge, or gauze pad with tap water or reverse osmosis and thoroughly wipe the ECG cable to remove any debris and all traces of the cleaning solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

Manual Disinfection: Prepare the disinfectant solution according to the manufacturer's instructions. Only disinfect clean products that have dried slightly. Saturate a synthetic felt or sponge pad with the disinfectant solution. Wipe all surfaces of the ECG cable. Wash the pad in the disinfectant solution and repeat wiping the cable. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 6 minutes. Moist a clean cloth, sponge, or gauze pad with reverse osmosis or distilled water and thoroughly wipe the ECG cable to remove any residual debris and all traces of disinfectant solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

ECG cable, especially metallic parts of the plugs and connector, should never be immersed in cleaning, disinfecting, or rinsing solutions or placed in an ultrasonic cleaning device. Soaking will reduce the life of the ECG cable by oxidation of metal parts and hardening of the cable jacket. Avoid using excessive force when cleaning or disinfecting by hand which may damage the internal wires of the cable and reduces the product life. Please do not clean with hard or frictional materials, abrasive cleaners, or solvents. Cleaning agents containing phenol or chloride are not suitable. The ECG cable should be completely dry before use, do not use wet ECG cables. These instructions were validated using the below detergents and disinfectants.

Cleaning agent (manual cleaning): Enzo® / Cidezyme®, enzymatic detergent, Johnson&Johnson (add 8ml enzymatic detergent per 1 liter distilled water, for cable with dried-on organic matter use 16 ml enzymatic detergent per 1 liter distilled water/ warm water)

Disinfectant (manual disinfection): Cidex OPA®, 0.55% ortho-phthalaldehyde solution, Johnson&Johnson (at a minimum temperature of 20°C/68°F). Reverse osmosis/ distilled water.

Caution: During the cleaning and disinfection of ECG cables, disposable gloves, protective eyewear, and a filtration mask should be used to reduce the risk of transmission of infectious agents by splashing or creating aerosols.

Warning: FMT ECG cables are sold NON-STERILE. Clean and disinfect the cables before the first and every use.

Warning: Before cleaning or disinfecting the cable, disconnect it from the ECG device.

Warning: ECG cable should be cleaned of overburden prior to disinfection to improve the effectiveness.

Warning: Do not sterilize ECG cables and leadwires by autoclave, radiation, or steam.

Warning: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents. Do not use bleach on electrical contacts or connectors.

Environmental Requirements:

Operating Temperature : 5°C - 45°C (41°F - 113°F)

Storage Temperature : 0°C - 50°C (32°F - 122°F)

Relative Humidity : 20 - 80% (not condensing)

During storage, the product should be protected from sunlight. It is recommended that the product be stored in its original package until the first use. Used cables should be loosely coiled in 10 to 15 cm diameter (4 to 6 inches) and stored individually in a protective container or packaging.

Compatibility: In order to ensure compatibility, the ECG cable should only be used with the equipment they have been designed for and is specified in the label on the product's primary packaging. For product models and compatible devices, please refer to the catalogue at metkomedical.com/PDF/02ECGCablesforPatientMonitoring.pdf

Safety:

Degree of protection from electric shocks: Type CF

Degree of protection against the ingress of water: Keep dry

The FMT ECG cables are sold NON-STERILE.

The FMT ECG cables are not made with natural rubber latex and PVC.

The FMT ECG cables have not been evaluated for safety in the MR environment. Scanning a patient who has this device may result in patient injury.

For an explanation of symbols, refer to the separate leaflet "Description of Symbols" located within the product package.

Reporting of Serious Incidents: Any serious incident related to the use of this product should be reported to both the manufacturer and the health authority/competent authority where the product is installed. Either contact your local representative or report to: metko@metkomedical.com

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Warranty: FMT ECG cables are under six (6) months warranty against material and workmanship defects from the original purchase date. In the warranty period, METKO will be responsible for repairing or changing the cable free of charge if the defect is proven. This warranty does not extend to any product that has been subject to misuse, wrong disinfection, neglect, or accident, or that has been damaged by causes external to the product, or that has been used in violation of the operating instructions supplied with the product. The product life is 3 years from the production date.

FMT® is a registered trademark of Metko A.Ş.

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