Indications: FMT reusable pressure infusion cuffs increase pressure on I.V. bags to rapidly infuse fluids during irrigation and infusion applications. They are intended for use by qualified medical personnel in hospitals and hospital-type facilities.

Contraindications: No known contraindications to the use of the product.

Adverse Reactions: No known adverse reactions to the use of the product.

Clinical benefits: Enables pressurization of IV fluid in an IV bag.

Product Description: The reusable pressure infusion cuff consists of a bag with a bladder, an envelope for holding the IV fluid bag, a manometer, and tubing with an inflation bulb and an air valve. Once the fluid bag is loaded in the envelope, the bag will then be inflated to the desired pressure. The parts mentioned in the use of the reusable pressure infusion cuff are given in the picture below. Infusion cuff loop (1), cuff hook (2), tubing (3), manometer (4), inflation bulb (5), air valve thumbwheel (6) and pinch clamp (7). The choice of the appropriate cuff size for the respective irrigation and infusion solution depends on the individual case and is the responsibility of the user. For optimal pressure transmission, the bag has to be completely covered by the inflated cuff.

- FMT-PI/05**, Reusable pressure infusion cuff, pocket type, 500 ml
- FMT-PI/10**, Reusable pressure infusion cuff, wrap-around type, 1000ml
- FMT-PI/10S**, Reusable pressure infusion cuff, pocket type, 1000ml
- FMT-PI/30**, Reusable pressure infusion cuff, pocket type, 3000ml

Directions for Use:

For FMT-PI/10 1000ml pressure infusion cuff:

- 1- Check the connections between the bulb, manometer, and tubing for air leakage.
- 2- Hang the pressure infusion cuff onto the IV pole by loop (1) located at the top of the cuff.
- 3- Open the mesh panel by removing the Velcro over the backside of the infusion cuff.
- 4- Hang the IV fluid bag on the cuff hook (2) located under the loop. Guide the IV tubing directly downwards.
- 5- Wrap the mesh panel firmly around the fluid bag and fasten it to Velcro at the backside of the infusion cuff (see Figure 2).
- 6- Open the pinch clamp (7) on the cuff delivery tubing.
- 7- Turn the thumbwheel of the air valve (5) clockwise to close the air valve vent.
- 8- Inflate the infusion cuff to the desired pressure by pumping the inflation bulb.
- 9- To maintain the pressure leave the thumbwheel valve fully clockwise turned position and lock the pinch clamp. Do not forget to open the pinch clamp for intermediate inflations.
- 10- To deflate, open the pinch clamp and turn the thumbwheel valve counter-clockwise to open the air vent.

For FMT-PI/10S, FMT-PI/05, FMT-PI/30 1000ml, 500ml, and 3000ml pressure infusion cuffs:

- 1- Check the connections between the bulb, manometer, and tubing for air leakage.
- 2- Hang the pressure infusion cuff onto IV pole by loop (1) located at the top of the cuff.
- 3- Insert the IV bag between the mesh and the bladder. (For 3000ml infusion cuffs, open the zipper at the top of the infusion cuff before inserting the IV bag and close the zipper after hanging the IV bag.)
- 4- Hang the IV fluid bag on the cuff hook (2) located under the loop. Guide the tubing directly downwards.
- 5- Open the pinch clamp (7) on the delivery tubing.
- 6- Turn the thumbwheel of the air valve (5) clockwise to close the air valve vent.
- 7- Inflate the infusion cuff to the desired pressure by pumping the inflation bulb.
- 8- To maintain the pressure leave the thumbwheel valve fully clockwise turned position and lock the pinch clamp. Do not forget to open the pinch clamp for intermediate inflations.
- 9- To deflate, open the pinch clamp and turn the thumbwheel valve counter-clockwise to open the air vent.

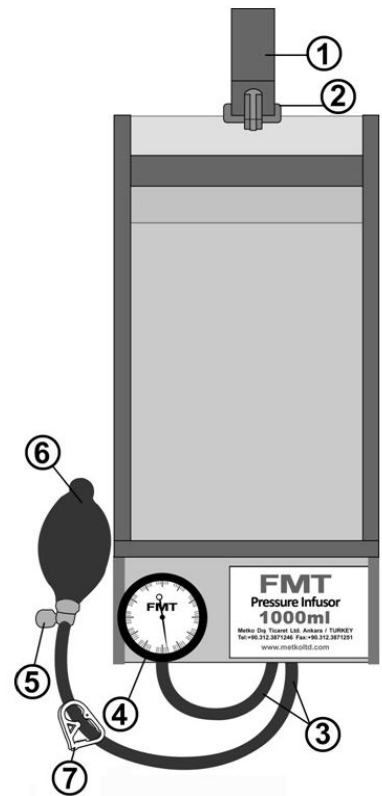
Figure 1: Infusion cuff parts

For replacing the bladder remove the bulb and manometer from the tubes first. Then open the zipper at the rear of the infusion cuff and pull the bladder out. Insert the new bladder into the cuff pocket and guide the tubing through the openings at the bottom of the pocket. Close the zipper and attach the bulb to the long tube and the manometer to the short tube.

The infusion cuff and its accessories do not require any specific maintenance. To test the manometer, remove the tube and hold it vertically. If the pointer stops at zero on the scale, the manometer is accurately adjusted. If it does not stop at zero, it should be replaced.

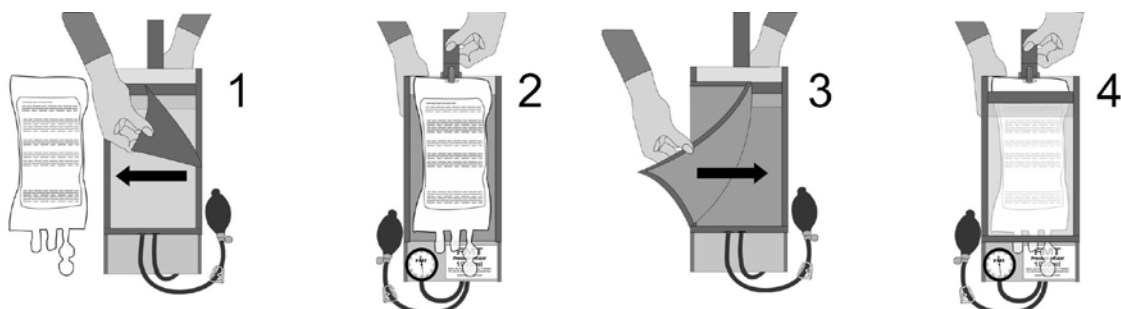
Figure 2: Application of IV fluid bag to the wrap-around type FMT-PI/10 1000ml pressure infusion cuff

Warnings:

- 1- Prior to infusion it is recommended that the IV administration set be properly primed and residual air removed from the fluid bag and tubing to minimize the risk of air embolism.
- 2- Never reuse partly used IV fluid bags in combination with a pressure infusion cuff.
- 3- Make sure that the pressure gauge is clear of any obstruction and allowed to move freely.
- 4- Check the infusion cuff pressure periodically during the infusion process.
- 5- The maximum pressure for the cuff must never exceed 300 mm Hg to prevent cuff damage.
- 6- Do not connect the infusion cuff to pressure-generating units.

- 7- During manual use, the cuff pressure must be re-adjusted using the inflation bulb to ensure compression.
- 8- Rough handling and abuse may affect the gauge accuracy.
- 9- Do not sterilize pressure infusion cuffs by autoclave, radiation, steam, or ethylene oxide (EtO).
- 10- The aneroid gauge should be verified at least once a year against a mercury manometer.
- 11- Pressure infusion cuffs should be connected and activated by qualified personnel only.
- 12- Do not apply external pressure against the cuff and avoid contact with the cuff during infusion.
- 13- Do not modify the pressure infusion cuff in any way. Modifications may affect patient safety, performance, and accuracy and void the warranty.
- 14- Do not use damaged pressure infusion cuffs. Dispose of damaged pressure infusion cuffs according to hospital disposal practices and local environmental and waste disposal laws and regulations.
- 15- Before using the pressure infusion cuffs, read the patient monitor's instruction manual for further information and warnings.

Manual Cleaning: The reusable pressure infusion cuffs (manometer, bulb, tubing, and infusion cuff) can be cleaned when necessary. Remove heavy debris from the pressure infusion cuff by wiping with an appropriate, moist, lint-free cloth, sponge, or gauze pad. Prepare the detergent solution with reverse osmosis or distilled water according to the manufacturer's instructions and take the solution into the spray bottle. Spray the detergent solution liberally on the cuff until saturated, and at least allow a contact time of 2 minutes. Wipe smooth surfaces with a soft cloth. Use a soft plastic cleaning brush for adhered particles or dried secretions. If visible contamination is still present on the cuff at the end of the cleaning repeat the cleaning process. Moist a clean cloth, sponge, or gauze pad with tap water or reverse osmosis and thoroughly wipe the cuff to remove any debris and all traces of the cleaning solution. Dry manually by using a lint-free cloth or absorbent paper. Infusion cuff can be hand washed up to 30°C after removing the bulb, manometer, and bladder. Frequent laundering will affect the lifetime of the infusion cuff. The pressure infusion cuffs should be completely dry before use, do not use wet cuffs. These instructions were validated using the below detergents and disinfectants.

Cleaning agent (manual cleaning): Enzol® / Cidezyme®, enzymatic detergent, Johnson&Johnson (add 8ml enzymatic detergent per 1 liter distilled water, for cable with dried-on organic matter use 16 ml enzymatic detergent per 1 liter distilled water/ warm water)
Reverse osmosis/ distilled water.

Caution: During the cleaning and disinfection of pressure infusion cuffs, disposable gloves, protective eyewear, and a filtration mask should be used to reduce the risk of transmission of infectious agents by splashing or creating aerosols.

Warning: FMT reusable pressure infusion cuffs are sold NON-STERILE.

Warning: Do not immerse the bulb, manometer, and bladder of the infusion cuff in any kind of liquid.

Warning: Do not sterilize pressure infusion cuffs by autoclave, radiation, or steam, and do not press with a hot iron.

Warning: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents.

Environmental Requirements:

Operating Temperature : 5°C - 45°C (41°F - 113°F)

Storage Temperature : 0°C - 50°C (32°F - 122°F)

Relative Humidity : 20 - 80% (not condensing)

During storage, the products should be protected from sunlight. It is recommended to store the product in its original package until the first use.

Compatibility: FMT reusable pressure infusion cuffs are intended to be used with 500 ml, 1000 ml, and 3000 ml IV fluid bags, depending on the appropriate size selected.

Safety:

Degree of protection from electric shocks: Out of scope

Degree of protection against the ingress of water: Keep dry

FMT reusable pressure infusion cuffs are sold NON-STERILE.

FMT reusable pressure infusion cuffs are not made with natural rubber latex and PVC.

Refer to the separate leaflet "Description of Symbols" located within the product package for the explanation of symbols.

Reporting of serious incidents: Any serious incident related to the use of this product should be reported to both the manufacturer and the health authority/competent authority where the product is installed. Either contact your local representative or report to: metko@metkomedical.com

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Warranty: FMT reusable pressure infusion cuffs are under six (6) months warranty against material and workmanship defects from the original purchase date. In the warranty period, METKO will be responsible for repairing or replacing the cuff free of charge if the defect is proven. This warranty does not cover any product that has been subjected to misuse, improper cleaning, neglect, or accident and that has been damaged by causes external to the product or that has been used in violation of the operating instructions supplied with the product. The product life is 3 years from the production date.

FMT® is a registered trademark of Metko A.Ş.

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