Indications: The FMT reusable invasive blood pressure (IBP) cables and adaptors are used to provide the electrical connection between the IBP transducer applied to the patient and the patient monitor, where blood pressure is continuously monitored invasively. Reusable IBP cables and adaptors are intended to be used with appropriate IBP monitoring devices by trained and qualified medical personnel in hospitals and hospital-type facilities.

Contraindications: No known contraindications to the use of the product.

Adverse Reactions: No known adverse reactions to the use of the product.

Clinical Benefits: Enables invasive blood pressure monitoring with patient monitoring equipment.

Product Description: The FMT IBP cable has a connector on one end that is compatible with the IBP connector on the patient monitor and a connector on the other end that is compatible with the connector on the IBP transducer. The FMT IBP adaptor connects IBP cables with different device-side connectors to patient monitors with different IBP connectors.

IBP-XX/XXX, Reusable IBP Cable, (XX shows the compatible disposable transducer number, XXX shows compatible patient monitor number).

IBP-ADP/XXX, Reusable IBP Adaptor, (XXX shows adaptor number).

Directions for Use:

- 1- Before use, check that the product is intact and clean.
- 2- Check product compatibility with existing IBP transducer and/or patient monitor before use.
- 3- Connect the IBP transducer, IBP cable, and patient monitor according to hospital protocol for pressure monitoring procedures.
- 4- Please refer to the IBP transducer and patient monitor user instructions of use for detailed information on IBP monitoring.

Warnings:

- 1- All IBP cables are designed for use with specific IBP transducers and patient monitors. Verify the compatibility of the IBP transducer, patient monitor, and all functions properly before use, otherwise degraded performance and/or patient injury can result.
- 2- Before each use, visually inspect the IBP cables. The IBP cables should be free of visible defects and damage. If there are breaks, scratches, or cracks on the cable jacket or plastics insulation of the connector, or there are visibly bent, damaged pins of the connector discontinue using the cable. Never use a damaged IBP cable or one with exposed electrical circuitry. Even if there is no exposed electrical circuit, the scratches or cracks in the cable jacket or plastic insulation may cause the accumulation of pathogens and reduce the effectiveness of cleaning and disinfection of the product. Dispose of damaged IBP cables according to hospital disposal practices, and local environmental and waste disposal laws, and regulations.
- 3- Do not use IBP cables during Magnetic Resonance Imaging (MRI) scanning or in an MRI environment as it may result in physical harm.
- 4- Keep the IBP cable connectors as far as possible from electromagnetic sources such as the power transformer of an electromedical device. Also, avoid placing the IBP cables alongside the power cord of an electromedical device. Strong electromagnetic fields may adversely interfere with the function of an IBP cable.
- 5- As with all medical equipment, carefully route cables to reduce the possibility of patient entanglement or strangulation.
- 6- To prevent shock hazards do not connect IBP cables to any power outlet.
- 7- To avoid a shock hazard and interface from nearby electrical equipment, keep IBP cables away from grounded metal and other electrical equipment.
- 8- When connecting and disconnecting the IBP cables pull on the connector itself, not on the cable.
- 9- Do not modify the IBP cables in any way. Modifications may affect patient safety, performance, and accuracy and void the warranty.
- 10- Mishandling of the IBP cables could result in damage to internal wires or loss of electrical isolation. Do not tie the IBP cables in knots or bunch tightly. Do not wrap IBP cables around equipment to avoid damaging internal wires. Do not leave the IBP cables dangling from the instrument. Keep IBP cables away from moving parts that might pinch or crush them. When not in use, loosely coil the cable to a bend diameter of at least 10 cm and store it at room temperature.
- 11- Mating connectors with force without proper alignment may damage the connectors and disrupt the electrical continuity.
- 12- For further information and warnings read the instruction manual of the patient monitor before using the IBP cables.

Manual Cleaning and Disinfection:

Manual Cleaning: The IBP cable and adaptor must be cleaned and disinfected immediately after each use. Remove heavy debris from the IBP cable and adaptor by wiping it with an appropriate, moist lint-free cloth, sponge, or gauze pad. Prepare the detergent solution with reverse osmosis or distilled water according to the manufacturer's instructions. Saturate a synthetic felt or sponge pad with the cleaning solution. Wipe all surfaces of the IBP cable and adaptor until all visible contamination is removed. Wash the pad in the cleaning solution and repeat wiping the cable. Use a plastic cleaning brush for adhered particles or dried secretions. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 2 minutes. If visible contamination is still present on the cable at the end of the cleaning repeat the cleaning process. Moist a clean cloth, sponge, or gauze pad with tap water or reverse osmosis and thoroughly wipe the IBP cable and adaptor to remove any debris and all traces of the cleaning solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

Manual Disinfection: Prepare the disinfectant solution according to the manufacturer's instructions. Only disinfect clean products that have dried slightly. Saturate a synthetic felt or sponge pad with the disinfectant solution. Wipe all surfaces of the IBP cable and adaptor. Wash the pad in the disinfectant solution and repeat wiping the cable. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 6 minutes. Moist a clean cloth, sponge, or gauze pad with reverse osmosis or distilled water and thoroughly wipe the IBP cable and adaptor to remove any residual debris and all traces of disinfectant solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

IBP cable and adaptor, especially metallic parts of the connector should never be immersed in cleaning, disinfecting, or rinsing solutions, or placed in an ultrasonic cleaning device. Soaking will reduce the life of the IBP cable and adaptor by oxidation of metal parts and hardening of the cable jacket. Avoid using excessive force when cleaning or disinfecting by hand which may damage the internal wires of the cable and reduces the product life. Please do not clean with hard or frictional materials, abrasive cleaners, or solvents. Cleaning agents containing phenol or chloride are not suitable. The IBP cable and adaptor should be completely dry before use, do not use wet IBP cable and adaptor. These instructions were validated using the below detergents and disinfectants.

Cleaning agent (manual cleaning): Enzol® / Cidezyme®, enzymatic detergent, Johnson & Johnson (add 8ml enzymatic detergent per 1 liter distilled water, for cable with dried-on organic matter use 16 ml enzymatic detergent per 1 liter distilled water/ warm water)

Disinfectant (manual disinfection): Cidex OPA®, 0.55% ortho-phthalaldehyde solution, Johnson & Johnson (at a minimum temperature of 20°C/68°F).

Reverse osmosis/ distilled water.

Caution: During the cleaning and disinfection of the IBP cables and adaptors disposable gloves, protective eyewear, and filtration mask should be used to reduce the risk of transmission of infectious agents by splashing or the creation of aerosols.

Warning: FMT IBP cables and adaptors are sold NON-STERILE. Clean and disinfect the cables before the first and every use.

Warning: Before cleaning or disinfecting the cable, disconnect it from the patient monitor.

Warning: IBP cables and adaptors should be cleaned of overburden prior to disinfection to improve the effectiveness.

Warning: Do not sterilize the IBP cables and adaptors by autoclave, radiation, or steam.

Warning: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents. Do not use bleach on electrical contacts or connectors.

Environmental Requirements:

Operating Temperature : 5°C - 45°C (41°F - 113°F)

Storage Temperature : 0°C - 50°C (32°F - 122°F)

Relative Humidity : 20 - 80% (not condensing)

During storage, the product should be protected from sunlight. It is recommended to store the product in its original package until the first use. Used cables should be loosely coiled in 10 to 15 cm diameter (4 to 6 inches) and must be stored individually in a protective container or sterile packaging.

Compatibility: In order to ensure compatibility, the IBP cable and adaptor should only be used with the equipment they have been designed and is specified on the label on the product's primary packaging. For product models and compatible devices, please refer to the catalog at metkomedical.com/PDF/09IBPCablesandAdapters.pdf

Safety:

Degree of protection from electric shocks: type BF

Degree of protection against the ingress of water: Keep dry

FMT IBP cables and adaptors are sold NON-STERILE.

FMT IBP cables and adaptors are not made with natural rubber latex and PVC.

Refer to the separate leaflet "Description of Symbols" located within the product package for the explanation of symbols.

Reporting of serious incidents: Any serious incident related to the use of this product should be reported to both the manufacturer and the health authority/competent authority where the product is installed. Either contact your local representative or report to: metko@metkomedical.com

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Warranty: FMT IBP cables and adaptors are under twelve (12) months warranty against material and workmanship defects from the date of original purchase. In the warranty period, METKO will be responsible for repairing or changing the cable free of charge if the defect is proven. This warranty does not extend to any product that has been subject to misuse, wrong disinfection, neglect, or accident, or that has been damaged by causes external to the product, or that has been used in violation of the operating instructions supplied with the product. The product life is 3 years of the production date.

FMT® is a registered trademark of Metko A.Ş.

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