Indications: FMT disposable pressure infusion cuffs (Pressure Infusor) are used to increase pressure on I.V. bags for the rapid infusion of fluids during irrigation and infusion applications. Pressure infusion cuffs are intended to be used by qualified medical personnel in hospitals and hospital-type facilities.

Contraindications: No known contraindications to the use of the product.

Adverse Reactions: No known adverse reactions to the use of the product.

Clinical benefits: Enables pressurization of IV fluid in an IV bag.

Product Description: The disposable pressure infusion cuff consists of a bag with a bladder, an envelope for holding the IV fluid bag, a manometer, tubing with an inflation bulb, and an air valve/stopcock. Once the fluid bag is loaded in the envelope, the bag will then be inflated to the desired pressure. The parts mentioned in the use of the disposable pressure infusion cuff are given in the picture below. Infusion cuff hanger eyelet (1), Velcro hook strip (2), circular Velcro loop (3), manometer (4), inflation bulb (5), and air valve/stopcock (6). The choice of the appropriate cuff size for the respective irrigation and infusion solution depends on the individual case and is the responsibility of the user. For optimal pressure transmission, the bag has to be completely covered by the inflated cuff.

FMT-DPI/05, Disposable pressure infusion cuff with stopcock valve, 500 ml
FMT-DPI/10, Disposable pressure infusion cuff with stopcock valve, 1000 ml

Directions for Use:

- 1- Check the connections between the bulb, manometer, and tubing for air leakage.
- 2- Hang the pressure infusion cuff on to IV pole by the eyelet (1) located at the top of the cuff.
- 3- Insert the IV bag between the mesh and the pressure cuff.
- 4- Hang the IV fluid bag to the infusion cuff by feeding the Velcro hook strip (2) through the top opening of the IV bag and fastening it to the circular Velcro loop (3). Guide the tubing direct downwards.
- 5- Turn the air valve/stopcock (6) handle to right towards the open stopcock vent (see Figure 2).
- 6- Inflate the infusion cuff by pumping the inflation bulb (5) until the piston gauge reaches the proper pressure (300 mmHg) indicated in the green portion on the manometer. Check the pressure from the manometer (piston gauge) (4). Disposable pressure infusion cuffs are equipped with a 325 mmHg pressure relief valve that prevents over inflation. Once 325 mmHg is reached, the manometer will indicate red on the manometer along with the word "caution" and pressure will begin to release from the valve. Pressure will continue to release from the valve until pressure decreases below 325 mmHg.
- 7- To maintain the pressure turn the air valve/stopcock handle up and away from the inflation bulb.
- 8- To deflate and release the pressure in the cuff, turn the air valve/stopcock handle straight down towards the inflation bulb.

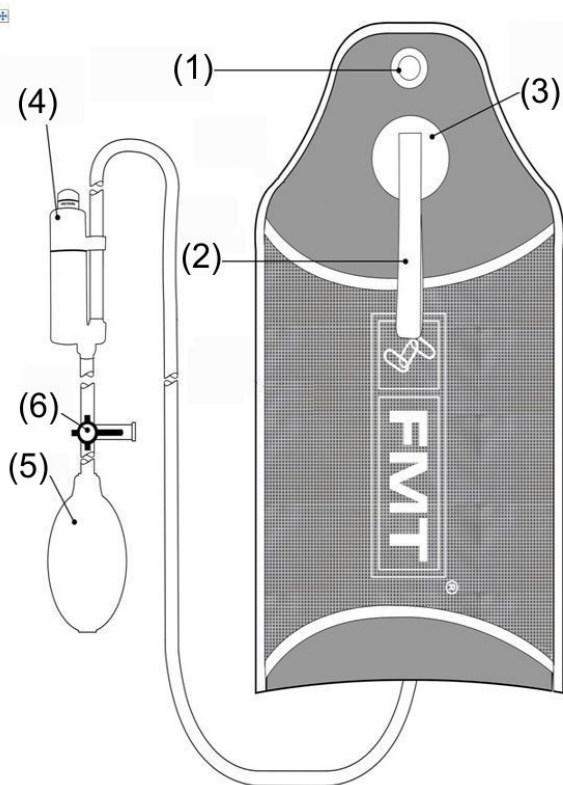
Figure 1: Disposable Infusion Cuff

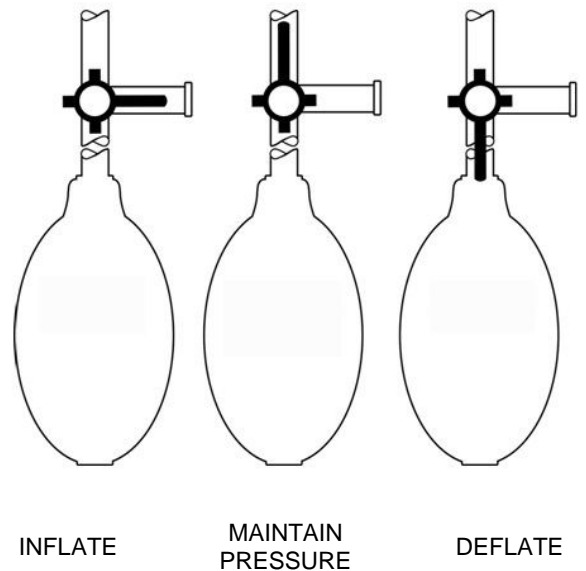
Figure 2: Air valve/stopcock handle positions

Warnings:

- 1- Prior to infusion it is recommended that the IV administration set be properly primed and residual air removed from the fluid bag and tubing to minimize the risk of air embolism.
- 2- Never reuse partly used IV fluid bags in combination with a pressure infusion cuff.

- 3- Disposable infusion cuffs are restricted for patients in isolation. Upon cessation of isolation, the infusion cuffs must be discarded. If the patient requires ongoing invasive monitoring a reusable infusion cuff should be used.
- 4- The patient's name should be written on the disposable infusion cuff using a permanent marker.
- 5- Make sure that the pressure gauge is clear of any obstruction and allowed to move freely.
- 6- Check the infusion cuff pressure periodically during the infusion process.
- 7- The maximum pressure for the cuff must never exceed 300 mmHg to prevent cuff damage.
- 8- Do not connect the infusion cuff to pressure-generating units.
- 9- During manual use the cuff pressure must be re-adjusted by means of the inflation bulb, to assure compression.
- 10- Rough handling and abuse may affect the gauge accuracy.
- 11- Do not sterilize pressure infusion cuffs by autoclave, radiation, steam, or ethylene oxide (EtO).
- 12- Pressure infusion cuffs should be connected and activated by qualified personnel only.
- 13- Do not apply external pressure against the cuff and avoid contact with the cuff during infusion.
- 14- Do not modify the pressure infusion cuff in any way. Modifications may affect patient safety, performance, and accuracy and void the warranty.
- 15- Do not use damaged pressure infusion cuffs or reuse them. Dispose of damaged pressure infusion cuffs according to hospital disposal practices, local environmental and waste disposal laws, and regulations.
- 16- For further information and warnings read the instruction manual of the patient monitor before using the pressure infusion cuffs.

Cleaning and Disinfection: Disposable pressure infusion cuffs are intended for single-patient use and should be disposed of after use, so do not attempt to clean, disinfect or sterilize them.

Environmental Requirements:

Operating Temperature : 5°C - 45°C (41°F - 113°F)

Storage Temperature : 0°C - 50°C (32°F - 122°F)

Relative Humidity : 20 - 80% (not condensing)

During storage, the products should be protected from sunlight. It is recommended to store the product in its original package until the first use.

Compatibility: FMT disposable pressure infusion cuffs are intended to use with 500 ml and 1000 ml IV fluid bags depending on the selection of the appropriate size.

Safety:

Degree of protection from electric shocks: Out of scope

Degree of protection against the ingress of water: Keep dry

FMT disposable pressure infusion cuffs are sold NON-STERILE.

FMT disposable pressure infusion cuffs are not made with natural rubber latex.

Refer to the separate leaflet "Description of Symbols" located within the product package for the explanation of symbols.

Reporting of serious incidents: Any serious incident related to the use of this product should be reported to both the manufacturer and the health authority/competent authority where the product is installed. Either contact your local representative or report to: metko@metkomedical.com

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Warranty: FMT disposable pressure Infusion cuffs are under three (3) months warranty against material and workmanship defects from the original purchase date. In the warranty period, METKO will be responsible for repairing or replacing the cuff free of charge if the defect is proven. This warranty does not cover any product that has been subjected to misuse, neglect, or accident and that has been damaged by causes external to the product, or that has been used in violation of the operating instructions supplied with the product. The product life is 3 years of the production date.

FMT® is a registered trademark of Metko A.Ş.

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