Indications: The fiber optic cable is indicated for medical use only, where a compatible light source and a compatible endoscope is used.

Contraindications: No known contraindications to the use of the product.

Adverse Reactions / Events: No known adverse reactions or events to the use of the product.

Product Description: The universal light source cable is used for endoscopic applications to transmit light during endoscopic diagnosis and therapy or for other medical illumination applications when the properties and operating instructions are complied with. The light source cables are designed for use with halogen, Xenon, or LED-based sources of cold light that are utilized in medical applications that involve endoscopes, medical instruments, or microscopes. The universal light source cable is composed of an optical glass fiber bundle in diameter of 4.8 mm, 3.5 mm, or 2.5 mm, a silicone rubber sheath, silicone rubber strain relievers, and stainless steel connectors. High-intensity visible light is conducted to the instrument through flexible light source cables. The use of the light source cable together with LASER light sources and/or HF devices is not permitted. FMT light source cables and adaptors are provided non-sterile and transport packaging is not sterilizable. Transport packaging must be removed and light cables and adaptors sterilized prior to use.

ULSCXXYY Standard Universal Light Source Cable (Both ends glued)

ULSCXXYYHP High Performance Universal Light Source Cable (Light source end hot fused)

ULSCXXYYEHP Extra High Performance Universal Light Source Cable (Both ends hot fused)

(XX Bundle Diameter, 2.5mm, 3.5mm and 4.8mm, YY Cable Length, 1.80m, 2.30m and 3.0m)

LA2XX Light Source (light input) adaptor, **LA1XX** Instrument (light output) adaptor (XX compatible device number)

Instructions for Use: Before using the light source cable, check compatibility of the cable and interchangeable stainless steel adaptors with the light source and the instrument. Then fix the corresponding ends of the light source cable to the light source device and the instrument as required.

Warnings:

- 1- Before using this device, read the user manuals of the other devices involved in the medical procedure to ensure compatibility.
- 2- This device is a non-sterile product. It is the user's responsibility to clean and sterilize this device before the first use and after every subsequent use.
- 3- Inspect the device after unpacking it to ensure it was not damaged during shipment and storage.
- 4- Immediately before and after every use, the light source cable and adaptors must be examined for integrity, correct functioning, and completeness.
- 5- Take the device out of the service immediately if any of the device connectors fail or if a cut, puncture, or other damage is found in the outer sheath.
- 6- Avoid direct contact of the distal end with body tissue or flammable materials. The heated light output of the light source cable can burn skin tissue and, if allowed to contact flammable materials such as drapes, plastics, and papers, cause a fire.
- 7- Never use the light source cable in the presence of any combustible gas.
- 8- Never look into the beam of light or direct the distal end tip of the light source cable or distal end of the endoscope toward other people.
- 9- If this product is used on a patient with, or suspected of having Creutzfeldt - Jacobs Disease (CJD) or its variants, the product cannot be reused and must be destroyed due to the inability to reprocess or sterilize to eliminate the risk of cross-contamination.
- 10- Replace the cable if the light transmission is compromised or if 30% of the fiber optics are damaged.
- 11- Use the handle/strain reliever to connect and disconnect the light source cable, never pull the cable by the sheathing to disconnect.
- 12- Always handle the device with care. Do not bend or kink the light source cables. Never coil in a diameter smaller than 15 cm. Do not tie light source cables with rubber bands. Do not pull the fiber when kinks or knots are present. Carefully work out any tangles. Avoid contact with sharp or pointed objects. Avoid mechanical load (for example by stepping on it, loading/dropping weights on) of the connectors and fiber. Permanent damage to the internal fiber optics and reduced light transmission may result.
- 13- Since the end of the connector contains glass, be careful not to hit the end of the connector against anything hard such as the floor or a desk by dropping the optical fiber item.
- 14- Avoid any soiling of the connectors. It may damage the fiber end surface or reduce light transmission. Connectors should be cleaned before interconnection. A microscopic bit of dirt or contamination can damage the connector or degrade performance. In high-power systems, this dirt can act as a lens to focus the high power and actually 'burn' the interconnection. Always perform the cleaning procedure described in the cleaning section for the optical face of the connectors before connecting the light source unit. Whenever possible, inspect each connector before connecting it to its mating device. This can be done using a simple 100x illuminated microscope (fiberscope).
- 15- The user of this product should be thoroughly familiar with and trained in the use and care of the product.
- 16- Bring the light source cable to the decontamination area as soon as possible after use. Observe valid protective measures to prevent contaminating the environment.
- 17- Do not sterilize light source cables by radiation, heated air, and formaldehyde or ethylene oxide.
- 18- To reduce the risk of skin burns wait five (5) minutes after turning off the light source to disconnect the cable.
- 19- Do not wash the cable or spray liquid onto the end tips of the fiber optic cable immediately after being used. Wait five (5) minutes for the end tips to cool before allowing any part of the cable to come in contact with any liquids.
- 20- It is recommended that a spare light source cable is present at the surgical site to replace the primary cable in case any issues arise.
- 21- The cable is designed to be easily connected to the devices for which it was designed. Do not try to connect the cable with a device if mechanical resistance is observed. Make sure the cable connection is compatible with the light source and the endoscope before use.
- 22- The light source cable is not intended to be processed using alkaline cleaners (pH above 10). The device will be damaged by alkaline cleaners. Do not use synthetic detergents, oil-based soaps, or any other cleaner not designed and certified for medical use. The interaction of these products with the light source cable can cause a chemical reaction modifying the original structure of the cable materials.
- 23- Dispose of damaged light source cables according to local laws and regulations for medical waste.
- 24- Read the instructions for use of the light source and instrument for additional information and warnings.

Manual Cleaning and Disinfection:

Manual Cleaning: The universal light source cables must be cleaned and disinfected immediately after each use. Disassemble or unlock instruments and adaptors for cleaning where applicable. Remove heavy debris from the cable by wiping with an appropriate, moist lint-free cloth, sponge, or gauze pad. Prepare the detergent solution with reverse osmosis or distilled water according to the manufacturer's instructions. Saturate a synthetic felt or sponge pad with the cleaning solution. Wipe all surfaces of the cable until all visible contamination is removed. Wash the pad in the cleaning solution and repeat wiping the cable. Use a plastic cleaning brush for adhered particles or dried secretions. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 2 minutes. If visible contamination is still present on the cable at the end of the cleaning repeat the cleaning process. Moist a clean cloth, sponge, or gauze pad with tap water or reverse osmosis and thoroughly wipe the cable to remove any debris and all traces of cleaning solution. Dry manually by using a lint free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

Manual Disinfection: Prepare the disinfectant solution according to the manufacturer's instructions. Only disinfect clean products that have dried slightly. Saturate a synthetic felt or sponge pad with the disinfectant solution. Wipe all surfaces of the universal light source cable. Wash the pad in the disinfectant solution and repeat wiping the cable. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 6 minutes. Moist a clean cloth, sponge, or gauze pad with reverse osmosis or distilled water and thoroughly wipe the cable to remove any residual debris and all traces of disinfectant solution. Dry manually by using a lint free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

If there is liquid inside the cable sheathing do not re-process the cable. In the event that liquid is discovered in the cable classify the cable as a defective device and replace it immediately. Universal light source cable, especially metallic parts of the connectors should never be immersed in cleaning, disinfecting, or rinsing solutions, or placed in an ultrasonic cleaning device. Soaking will reduce the life of cable by oxidation of metal parts and hardening of the cable jacket. Avoid using

excessive force when cleaning or disinfecting by hand. Please do not clean with hard or frictional materials, abrasive cleaners, or solvents. Cleaning agents containing phenol or chloride are not suitable. The light source cable should be completely dry before use, do not use wet light source cables. These instructions were validated using the below detergents and disinfectants.

Cleaning agent (manual cleaning): Enzol® / Cidezyme®, enzymatic detergent, Johnson&Johnson (add 8ml enzymatic detergent per 1 liter distilled water, for cable with dried-on organic matter use 16 ml enzymatic detergent per 1 liter distilled water/ warm water)

Disinfectant (manual disinfection): Cidex OPA®, 0.55% ortho-phthalaldehyde solution, Johnson&Johnson (at a minimum temperature of 20°C/68°F).

Reverse osmosis/ distilled water.

Caution: During the cleaning and disinfection of light source cables disposable gloves, protective eyewear, and filtration mask should be used to reduce the risk of transmission of infectious agents by splashing or the creation of aerosols.

Note: For cleaning the optical faces of the connectors, use a pure grade of isopropyl alcohol on a clean lint-free tissue or lens-cleaning tissue off to wipe. After the wet scrub, wipe the surfaces with dry lint-free tissue. Do not leave the fluid to air dry.

Warning: FMT universal light source cables are sold NON STERILE. Clean and disinfect the cables before first and every use.

Warning: Before cleaning or disinfecting the universal light source cables, disconnect the cable from light source unit.

Warning: Light source cables should be cleaned of overburden prior to disinfection to improve the effectiveness.

Warning: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents.

Sterilization: The universal light source cable must be cleaned and disinfected prior to sterilization. When preparing the cable for sterilization, do not coil cable into a circle of less than 15cm (6") in diameter. For best sterilization performance and longest device life it is best to sterilize the cable separately. Other items in the sterilization tray may result in incomplete sterilization or damage to the fiber optic cable. Place the fiber optic cable in a tray approved for use with this type of device. Do not place the cable in a Tyvek pouch for Steam or STERRAD systems as the pouch may melt to the fiber optic cable and cause permanent damage to the device. Sterilization of the light source cables should be done by steam sterilization using the fractioned pre vacuumed method. The cables must be sterilized at a minimum of 134°C [273°F] and a maximum of 137°C [278°F] in saturated steam during a holding (exposure) time of at least 4 minutes to at most 20 minutes, then dried in vacuum for at least 20 minutes. When sterilizing several products at the same time in a steam autoclave, ensure that the maximum load capacity of the autoclave specified by the manufacturer is not exceeded. Sterilization must be performed in accordance with EN ISO 17665 (Sterilization of Medical Devices in Moist Heat).

Note: Before use, the universal light source cable must be cooled to room temperature. It is the sole responsibility of the user to safeguard and maintain the sterile condition of the cables after the sterilization process.

Note: The use of Tyvek / film sterilization packaging is not permissible for steam sterilization; only the use of paper / film sterilization packaging is permitted.

Warning: Do not use heated air or Flash sterilization or other procedures that are intended exclusively for use without sterilization packaging, radiation sterilization, formaldehyde, ethylene oxide or another chemical sterilization.

Environmental Requirements:

Operating Temperature : 5°C - 45°C (41°F - 113°F)

Storage Temperature : 0°C - 50°C (32°F - 122°F)

Relative Humidity : 20 - 80% (not condensing)

During the storage the products should be protected from sunlight. It is recommended to store the products in original packages until the first use. Used cables should be loosely coiled greater than 15 cm diameter (6 inches) and must be stored individually in a protective container. Do not stack cables in their original packaging more than 20 units tall. Do not stack other products on the cable container.

Compatibility: The universal light source cable must be used with the appropriate light source and instrument adaptors attached to ensure compatibility. For product models and compatible devices, please see the catalog at metkomedical.com/PDF/ENG/ULSC.pdf

Safety:

Degree of protection from electric shocks: type BF

Degree of protection against the ingress of water: IPX1

FMT universal light source cable and adaptor are not made with natural rubber latex.

FMT universal light source cable and adaptor are not made with PVC.

Refer to the separate leaflet "Description of Symbols" located within the product package for the explanation of symbols.

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Warranty: FMT universal light source cable and adaptor are under twelve (12) months warranty against all material and workmanship defects except for broken fiber from the date of original purchase. In warranty period, METKO will be responsible for repairing the cable or change the cable free of charge if the defect is proven. This warranty does not extend to any product that has been subject to misuse, wrong sterilization, neglect or accident; or that has been damaged by causes external to the product; or that has been used in violation of the operating instructions supplied with the product. The product life is 3 years of the production date.

FMT® is a registered trademark of Metko A.Ş.

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