Indications:

FMT AFT series reusable air flow temperature probe is indicated for continuous measurement of chamber outlet and airway temperatures and the measurement of the flow rate of the gas with the F&P MR850/HC550 humidifiers and RT-Series breathing circuits. The reusable airflow temperature probes are intended to be used with compatible humidifiers by trained and qualified medical personnel in hospitals and hospital-type facilities.

Contraindications:

No known contraindications to the use of the product.

Adverse Reactions:

No known adverse reactions to the use of the product.

Clinical benefits:

Enables temperature monitoring with humidifier.

Product Description:

Probes are accurate and interchangeable with any other probe with matching temperature/resistance characteristics at specified measurement temperatures. Within the measurement range of 25 - 45 °C, the accuracy for the probes is $\pm 0.3^{\circ}\text{C}$. The minimum measurement time for accurate temperature readings is 150 seconds. The temperature reaches a stabilized temperature in 5 minutes.


FMT-AFT/A1 (125 cm), for use with MR850/HC550 humidifier and 0.7 m (28") RT Series breathing circuits

FMT-AFT/A2 (165 cm), for use with MR850/HC550 humidifier and 1.1 m (44") RT Series breathing circuits

FMT-AFT/A3 (185 cm), for use with MR850/HC550 humidifier and 1.3 m (52") RT Series breathing circuits


FMT-AFT/A4 (205 cm), for use with MR850/HC550 humidifier and 1.5 m (60") RT Series breathing circuits

FMT-AFT/A5 (235 cm), for use with MR850/HC550 humidifier and 1.8 m (72") RT Series breathing circuits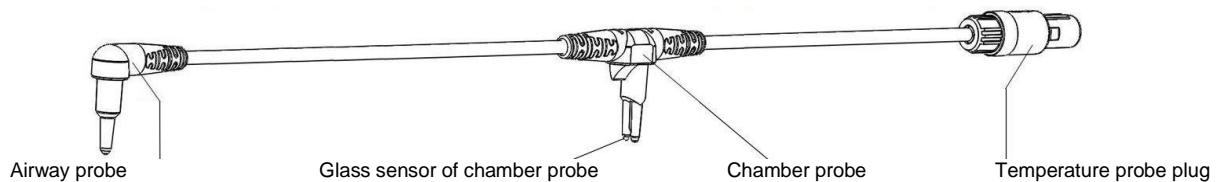

Directions for Use:

- 1- Before use, check that the product is intact and clean. Inspect the temperature probe and cable for wear, breakage, or fraying. Also, check that the glass tip of the chamber probe has not been damaged. Replace probe if necessary.
- 2- Remove the transparent protection hose over the glass sensor of the chamber probe. Inspect the chamber probe's glass tip for deposits or foreign material. Clean the probe as required.
- 3- Follow the humidifier manufacturer's instructions of use and your hospital's protocols to apply the airflow temperature probe to the humidifier and the breathing circuit.
- 4- Connect the temperature probe plug to the temperature socket (blue) on the humidifier base until an audible click is heard.
- 5- Insert the chamber probe and airway probe into the corresponding probe ports of the breathing circuit. Make sure the chamber probe is correctly located in its key-way and that both probes are pushed home completely. The probe lead can be restrained using breathing circuit clips.

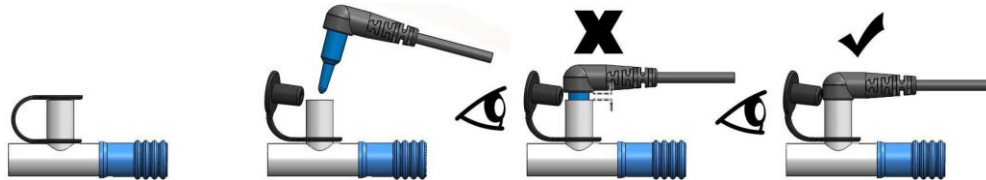Warnings:

- 1- All temperature probes are designed for use with specific monitors, instruments, or electronic thermometers. Verify the compatibility of the device and probe and all functions including alarming perform properly before use, otherwise degraded performance, accuracy, and/or patient injury can result.
- 2- Application of temperature probes should be done by trained medical personnel following established procedures.
- 3- Ensure both temperature probe tips are correctly and securely fitted. Failure to do so may result in temperatures in excess of 41°C being delivered to the patient.
- 4- This probe is intended for use with F&P and compatible humidifiers. The use of probes with other incompatible devices may lead to inaccurate measurements.
- 5- Do not touch the glass tip of the chamber temperature probe during use as this may result in minor skin burns.
- 6- Ensure the black electrical connector is dry at all times. Failure to do so may damage the device.
- 7- Protect probe airway and chamber probe tips, especially sensitive glass sensor of chamber probe against any impact. Keep the transparent hose attached to the glass sensor when the probe is not in use.
- 8- Protect the airway temperature probe from radiant heat with Reflective Shield to improve humidifier performance and reduce circuit condensation with radiant infant warmers.
- 9- It is not recommended to use the probes within the range of MRI, CT, etc. applications.
- 10- Mishandling of the probes could result in damage to internal wires and loss of electrical isolation or improper temperature readings. Don't tie the probes in knots or bunch tightly. Don't wrap probes around equipment to avoid damaging internal wires. Don't leave the probes dangling from the instrument. When not in use, probes and leads should be loosely coiled and stored at room temperature.
- 11- Care must be taken to ensure that the probe cable is not subjected to mechanical stress, such as pulling, bending, and squeezing.
- 12- Do not use damaged probes. Dispose of damaged probes according to local laws and regulations for medical waste.
- 13- Never boil or autoclave temperature probe or clean with an ultrasonic cleaner. Do not immerse the probe in any liquid.
- 14- Do not use wet probes.
- 15- For further information and warnings read the instructions of use accompanying the humidifier.

Manual Cleaning and Disinfection:

Manual Cleaning:

The air temperature probes must be cleaned and disinfected immediately after each use. Remove heavy debris from the temperature probe by wiping with an appropriate, moist lint-free cloth, sponge, or gauze pad. Prepare the detergent solution with reverse osmosis or distilled water according to the manufacturer's instructions. Saturate a synthetic felt or sponge pad with the cleaning solution. Wipe all surfaces of the temperature probe until all visible contamination is removed. Wash the pad in the cleaning solution and repeat wiping the cable. Use a plastic cleaning brush for adhered particles or dried secretions. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 2 minutes. If visible contamination is still present on the probe at the end of the cleaning repeat the cleaning process. Moist a clean cloth, sponge, or gauze pad with tap water or reverse osmosis and thoroughly wipe the temperature probe to remove any debris and all traces of the cleaning solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

Manual Disinfection:

Prepare the disinfectant solution according to the manufacturer's instructions. Only disinfect clean products that have dried slightly. Saturate a synthetic felt or sponge pad with the disinfectant solution. Wipe all surfaces of the airflow temperature probe. Wash the pad in the disinfectant solution and repeat wiping the probe

cable. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 6 minutes. Moist a clean cloth, sponge, or gauze pad with reverse osmosis or distilled water and thoroughly wipe the temperature probe to remove any residual debris and all traces of disinfectant solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

Temperature probes, especially metallic parts of connectors should never be immersed in cleaning, disinfecting, or rinsing solutions, or placed in an ultrasonic cleaning device. Soaking will reduce the life of the temperature probe by oxidation of metal parts and hardening of the cable jacket. Avoid using excessive force when cleaning or disinfecting by hand that may damage the internal wires of the probe and reduces the product life. Please do not clean with hard or frictional materials, abrasive cleaners, or solvents. Cleaning agents containing phenol or chloride are not suitable. The temperature probe should be completely dry before use, do not use wet temperature probes. These instructions were validated using the below detergents and disinfectants.

Cleaning agent (manual cleaning): Enzo® / Cidezyme®, enzymatic detergent, Johnson&Johnson (add 8ml enzymatic detergent per 1 liter distilled water, for cable with dried-on organic matter use 16 ml enzymatic detergent per 1 liter distilled water/ warm water).

Disinfectant (manual disinfection): Cidex OPA®, 0.55% ortho-phthalaldehyde solution, Johnson&Johnson (at a minimum temperature of 20°C/68°F).

Reverse osmosis/ distilled water.

Caution: During the cleaning and disinfection of airflow temperature probes disposable gloves, protective eyewear, and filtration mask should be used to reduce the risk of transmission of infectious agents by splashing or the creation of aerosols.

Warning: FMT airflow temperature probes are sold NON-STERILE. Clean and disinfect the probes before first and every use.

Warning: Before cleaning or disinfecting the probe, disconnect it from the humidifier.

Warning: No automated method is recommended. Do not use dishwasher detergents.

Warning: Airflow Temperature probes should be cleaned of overburden prior to disinfection to improve the effectiveness.

Warning: Do not sterilize air temperature probes by autoclave, radiation, or steam.

Warning: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents. Do not use bleach on electrical contacts or connectors.

Environmental Requirements:

Operating Temperature : 5°C - 45°C (41°F - 113°F)

Storage Temperature : 0°C - 50°C (32°F - 122°F)

Relative Humidity : 20 - 80% (not condensing)

During storage, the products should be protected from sunlight. It is recommended to store the product in its original package until the first use. Used probes should be loosely coiled in 10 to 15 cm diameter (4 to 6 inches) and stored individually in a protective container or packaging.

Compatibility:

In order to ensure compatibility and claimed accuracy, the temperature probe should only be used with the equipment for which they have been designed and is specified in the label on the product's primary packaging. All airflow temperature probes are compatible with F&P MR850/HC550 humidifiers and other humidifiers using the same technology as these humidifiers. For product models and compatible devices, please refer to the catalog at metkomedical.com/PDF/04MedicalTemperatureProbes.pdf

Safety:

Degree of protection from electric shocks: type B

Degree of protection against the ingress of water: Keep dry

FMT reusable airflow temperature probes are sold NON-STERILE.

FMT reusable temperature probes are not made with natural rubber latex.

Refer to the separate leaflet "Description of Symbols" located within the product package for the explanation of symbols.

Reporting of serious incidents:

Any serious incident related to the use of this product should be reported to both the manufacturer and the health authority/competent authority where the product is installed. Either contact your local representative or report to: metko@metkomedical.com

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Warranty:

FMT reusable air flow temperature probes are under twelve (12) months warranty against material and workmanship defects from the date of original purchase. In the warranty period, METKO will be responsible for repairing or changing the probe free of charge if the defect is proven. This warranty does not extend to any product that has been subject to misuse, wrong disinfection, neglect, or accident; that has been damaged by causes external to the product, or that has been used in violation of the operating instructions supplied with the product. The product life is 3 years of the production date.

FMT® is a registered trademark of Metko A.Ş.

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