Indications: FMT SpO2 adaptor and extension cables are used to transmit electrical signals received from the reusable or disposable SpO2 sensor applied to the patient to the pulse oximeter or patient monitor for continuous noninvasive monitoring of functional oxygen saturation of arterial hemoglobin (SpO2) and pulse rate. SpO2 adaptor cables are intended to be used by qualified medical personnel in hospitals and hospital-type facilities. SpO2 adaptor cables can be used in infant, pediatric and adult patients, depending on the selection of the appropriate size disposable or reusable SpO2 sensor.

Contraindications: No known contraindications to the use of the product.

Adverse Reactions: No known adverse reactions to the use of the product.

Clinical Benefits: Enables SpO2 monitoring with patient monitoring equipment.

Product Description: FMT SpO2 adaptor and extension cables have different sensor-side connectors, device-side connectors, and connector wiring configurations for different pulse oximeter or patient monitor manufacturers. Therefore, the same appearance does not guarantee full compatibility with the product. Always consider the information on the product's label and/or primary packaging.

AEC-5XXX Reusable SpO2 adaptor & extension cable, where XXX shows the number for compatibility.

Directions for Use:

- 1- Before use, check that the product is intact and clean.
- 2- Connect the SpO2 sensor to the sensor side connector of the SpO2 adaptor cable.
- 3- Connect the SpO2 adaptor cable connector to the socket of the compatible pulse oximeter or patient monitor and check it for correct operation.
- 4- Apply the SpO2 sensor to the patient as indicated in the SpO2 sensor instructions for use and wait for SpO2 (%), pulse rate (PR, bpm), and plethysmographic pulse wave or signal to appear on the screen of the pulse oximeter.
- 5- Please refer to the pulse oximeter or patient monitor user instructions of use for detailed information on SpO2 monitoring.

Warnings:

- 1- All SpO2 adaptor cables are designed for use with a specific SpO2 sensor and pulse oximeter or patient monitor. Before use, verify the compatibility of the pulse oximeter, SpO2 sensor, and all functions, including alarming performance. Otherwise, degraded performance and/or patient injury can result.
- 2- Before each use, visually inspect the SpO2 adaptor cables. The SpO2 adaptor cables should be free of visible defects and damage. If there are breaks, scratches, or cracks on the cable jacket or plastic insulation of the connector, or there are visibly bent, damaged pins of the connector, discontinue using the cable. Never use a damaged adaptor cable or one with exposed electrical circuitry. Even if there is no exposed electrical circuit, scratches or cracks in the cable jacket or plastic insulation may accumulate pathogens and reduce the effectiveness of cleaning and disinfecting the product. Dispose of damaged adaptor cables according to hospital disposal practices and local environmental and waste disposal laws and regulations.
- 3- Do not use adaptor cables during Magnetic Resonance Imaging (MRI) scanning or in an MRI environment, as this may result in physical harm.
- 4- Keep the adaptor cable's connector as far as possible from electromagnetic sources, such as an electromedical device's power transformer. Also, avoid placing it alongside the device's power cord. Strong electromagnetic fields may adversely interfere with the adaptor and extension cable's function.
- 5- As with all medical equipment, carefully route cables to reduce the possibility of patient entanglement or strangulation.
- 6- To prevent shock hazards, do not connect the adaptor cable to any power outlet.
- 7- To avoid a shock hazard and interface with nearby electrical equipment, keep the adaptor and extension cable away from grounded metal and other electrical equipment.
- 8- When connecting and disconnecting the adaptor cables pull on the connector itself, not on the cable.
- 9- Do not modify the adaptor cables in any way. Modifications may affect patient safety, performance, and accuracy and void the warranty.
- 10- Mishandling of the adaptor cables could result in damage to internal wires or loss of electrical isolation. Do not tie the cables in knots or bunch them tightly. Do not wrap cables around equipment to avoid damaging internal wires. Do not leave the cables dangling from the instrument. Keep the cables away from moving parts that might pinch or crush them. When not in use, loosely coil the cables to a bend diameter of at least 10 cm and store it at room temperature.
- 11- Mating connectors with force without proper alignment may damage the connectors and disrupt the electrical continuity.
- 12- For further information and warnings, read the instruction manual of the pulse oximeter or patient monitor before using the adaptor cables.

Manual Cleaning and Disinfection:

Manual Cleaning: The SpO2 adaptor cables must be cleaned and disinfected immediately after each use. Remove heavy debris from the SpO2 adaptor cable by wiping it with an appropriate, moist, lint-free cloth, sponge, or gauze pad. Prepare the detergent solution with reverse osmosis or distilled water according to the manufacturer's instructions. Saturate a synthetic felt or sponge pad with the cleaning solution. Wipe all surfaces of the SpO2 adaptor & extension cable until all visible contamination is removed. Wash the pad in the cleaning solution and repeat wiping the cable. Use a plastic cleaning brush for adhered particles or dried secretions. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 2 minutes. If visible contamination is still present on the cable at the end of the cleaning repeat the cleaning process. Moist a clean cloth, sponge, or gauze pad with tap water or reverse osmosis and thoroughly wipe the SpO2 adaptor cable to remove any debris and all traces of the cleaning solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

Manual Disinfection: Prepare the disinfectant solution according to the manufacturer's instructions. Only disinfect clean products that have dried slightly. Saturate a synthetic felt or sponge pad with the disinfectant solution. Wipe all surfaces of the SpO2 adaptor & extension cable. Wash the pad in the disinfectant solution and repeat wiping the cable. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 6 minutes. Moist a clean cloth, sponge, or gauze pad with reverse osmosis or distilled water and thoroughly wipe the SpO2 adaptor & extension cable to remove any residual debris and all traces of disinfectant solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

SpO2 adaptor cable, especially metallic parts of the connector, should never be immersed in cleaning, disinfecting, or rinsing solutions or placed in an ultrasonic cleaning device. Soaking will reduce the life of the SpO2 adaptor cable by oxidation of metal parts and hardening of the cable jacket. Avoid using excessive force when cleaning or disinfecting by hand which may damage the internal wires of the cable and reduces the product's life. Please do not clean with hard or frictional materials, abrasive cleaners, or solvents. Cleaning agents containing phenol or chloride are not suitable. The SpO2 adaptor & extension cable should be completely dry before use, do not use wet SpO2 adaptor cables. These instructions were validated using the below detergents and disinfectants.

Cleaning agent (manual cleaning): Enzol® / Cidezyme®, enzymatic detergent, Johnson&Johnson (add 8ml enzymatic detergent per 1 liter distilled water, for cable with dried-on organic matter use 16 ml enzymatic detergent per 1 liter distilled water/ warm water)

Disinfectant (manual disinfection): Cidex OPA®, 0.55% ortho-phthalaldehyde solution, Johnson&Johnson (at a minimum temperature of 20°C/68°F).

Reverse osmosis/ distilled water.

Caution: During the cleaning and disinfection of the SpO2 adaptor cables, disposable gloves, protective eyewear, and a filtration mask should be used to reduce the risk of transmission of infectious agents by splashing or creating aerosols.

Warning: FMT SpO2 adaptor cables are sold NON-STERILE. Clean and disinfect the cables before the first and every use.

Warning: Before cleaning or disinfecting the cable, disconnect it from the pulse oximeter and the sensor.

Warning: SpO2 adaptor cables should be cleaned of overburden prior to disinfection to improve the effectiveness.

Warning: Do not sterilize the SpO2 adaptor & extension cables by autoclave, radiation, or steam.

Warning: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents. Do not use bleach on electrical contacts or connectors.

Environmental Requirements:

Operating Temperature : 5°C - 45°C (41°F - 113°F)

Storage Temperature : 0°C - 50°C (32°F - 122°F)

Relative Humidity : 20 - 80% (not condensing)

During storage, the product should be protected from sunlight. It is recommended to store the product in its original package until the first use. Used cables should be loosely coiled in 10 to 15 cm diameter (4 to 6 inches) and must be stored individually in a protective container or packaging.

Compatibility: To ensure compatibility, the SpO2 adaptor cable should only be used with the equipment for which they have been designed and is specified in the label on the product's primary packaging. For product models and compatible devices, please refer to the catalogue at [metkomedical.com/PDF/03PulseOximetry\(SpO2\)Sensors.pdf](http://metkomedical.com/PDF/03PulseOximetry(SpO2)Sensors.pdf)

Safety:

Degree of protection from electric shocks: type BF

Degree of protection against the ingress of water: IPX1

The FMT SpO2 adaptor & extension cables are sold NON-STERILE.

The FMT SpO2 adaptor & extension cables are not made with natural rubber latex and PVC.

The FMT SpO2 adaptor & extension cables have not been evaluated for safety in the MR environment. Scanning a patient who has this device may result in patient injury.

For an explanation of symbols, refer to the separate leaflet "Description of Symbols" located within the product package.

Reporting of Serious Incidents: Any serious incident related to the use of this product should be reported to both the manufacturer and the health authority/competent authority where the product is installed. Either contact your local representative or report to: metko@metkomedical.com

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Warranty: FMT SpO2 adaptor & extension cables are under twelve (12) months warranty against material and workmanship defects from the date of original purchase. In the warranty period, METKO will be responsible for repairing or changing the cable free of charge if the defect is proven. This warranty does not extend to any product that has been subject to misuse, wrong disinfection, neglect, or accident, or that has been damaged by causes external to the product or that has been used in violation of the operating instructions supplied with the product. The product life is 3 years from the production date.

FMT® is a registered trademark of Metko A.Ş.

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