Indications: FMT Reusable Non-Invasive Blood Pressure (NIBP) hoses are used to connect NIBP cuffs applied to the patient to the automated oscillometric NIBP devices to measure the patient's systolic and diastolic arterial pressure. NIBP hoses are intended to be used by qualified medical personnel and can be used in infant, pediatric and adult patients, depending on the selection of the appropriate size disposable or reusable cuff in hospitals and hospital-type facilities.

Contraindications: No known contraindications to the use of the product.

Adverse Reactions: No known adverse reactions to the use of the product.

Clinical Benefits: Enables NIBP monitoring with patient monitoring equipment.

Product Description: FMT reusable NIBP hoses are made of Thermoplastic Polyurethane (TPU). The single-hose or double-hose configuration is dependent on the automated NIBP monitoring system. The hose connects to the compatible NIBP cuff and NIBP monitor via connectors attached to both ends.

SBPXX/BPXX, Single NIBP hose, 3m

DBPXXXX/BPXXXX, Double NIBP hose, 3m

CEBPXX/BPXX, Coiled and Extendable Single NIBP hose, 3m (XX is the connector number.)

Directions for Use:

- 1- Before use, check that the NIBP cuff and NIBP hose are intact and clean. Replace the hose if it shows wear, tear, loosening, or breakage of the connectors that could cause air leakage. Do not use the hose if structural integrity is suspected.
- 2- Carefully inspect the connectors on both sides of the hose to ensure they are compatible with the connectors on the NIBP cuff and the NIBP monitor. Connect the hose to the NIBP monitor. Make sure the hose connector is properly attached to the monitor.
- 3- Apply the cuff to the patient according to your hospital protocol or as instructed in the instruction manual of the cuff.
- 4- Connect the cuff to the NIBP hose, ensuring the proper engagement of the connectors. Ensure that the connection is secure, that the tubing is not kinked, and that the hose is routed to avoid kinking or compression.
- 5- Start NIBP monitoring.

Warnings:

- 1- NIBP hoses are designed for use with compatible automated oscillometric NIBP devices to measure arterial pressure on the patient's upper arm. Verify the compatibility of the NIBP cuff, NIBP hose, and NIBP monitoring device all functions perform properly before use, otherwise degraded performance and/or patient injury can result.
- 2- Before each use, visually inspect the NIBP cuff and connected NIBP hose. The NIBP hoses should be free of visible defects and damage. Never use damaged, clogged, or visibly contaminated NIBP hoses. Dispose of damaged NIBP hoses according to hospital disposal practices and local environmental and waste disposal laws and regulations.
- 3- Do not connect the hose to intravascular fluid systems, as this could allow air to be pumped into a blood vessel, which could lead to serious patient injury.
- 4- Failure to properly connect the hose to the cuff or patient monitor will result in intermittent readings, inaccurate results, or no reading.
- 5- To avoid damaging the hose, always hold it by the connector rather than the hose when connecting or disconnecting either end.
- 6- Do not immerse the NIBP hose in any liquid. The fluid in the hose may damage the monitoring device.
- 7- Keep hoses away from moving parts that may pinch and crush them.
- 8- Avoid placing the cuff and hose on a patient so that the tube becomes kinked, compressed, or obstructed. This may cause inaccurate readings.
- 9- Do not modify the NIBP hoses in any way. Modifications may affect patient safety, performance, and accuracy and void the warranty.
- 10- As with all medical equipment, carefully route hoses to reduce the possibility of patient entanglement or strangulation.
- 11- Before using the NIBP hoses, read the instruction manual of the NIBP monitoring device or patient monitor for further information and warnings.

Manual Cleaning and Disinfection:

Manual Cleaning: The NIBP hoses must be cleaned and disinfected immediately after each use. Remove heavy debris from the NIBP hose by wiping it with an appropriate, moist, lint-free cloth, sponge, or gauze pad. Prepare the detergent solution with reverse osmosis or distilled water according to the manufacturer's instructions. Saturate a synthetic felt or sponge pad with the cleaning solution. Wipe all surfaces of the hose until all visible contamination is removed. Wash the pad in the cleaning solution and repeat wiping the hose. Use a plastic cleaning brush for adhered particles or dried secretions. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 2 minutes. If visible contamination is still present on the hose at the end of the cleaning repeat the cleaning process. Moist a clean cloth, sponge, or gauze pad with tap water or reverse osmosis and thoroughly wipe the hose to remove any debris and all traces of the cleaning solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

Manual Disinfection: Prepare the disinfectant solution according to the manufacturer's instructions. Only disinfect clean products that have dried slightly. Saturate a synthetic felt or sponge pad with the disinfectant solution. Wipe all surfaces of the hose. Wash the pad in the disinfectant solution and repeat wiping the hose. Wipe or brush all surfaces at least six strokes back and forth. At least allow a contact time of 6 minutes. Moist a clean cloth, sponge, or gauze pad with reverse osmosis or distilled water and thoroughly wipe the hose to remove any residual debris and all traces of disinfectant solution. Dry manually by using a lint-free cloth or absorbent paper and, in particular, for drying cavities, sterile compressed air.

NIBP hose, especially metallic parts of the connector, should never be immersed in cleaning, disinfecting, or rinsing solutions or placed in an ultrasonic cleaning device. Soaking will reduce the life of the hose by oxidation of metal parts. Care should be taken to prevent liquid from entering the hose tubing. The liquid in the tubing may affect blood pressure determination accuracy and damage automatic or manual monitors. Avoid using excessive force when cleaning or disinfecting by hand. Please do not clean with hard or

frictional materials, abrasive cleaners, or solvents. Cleaning agents containing phenol or chloride are not suitable. The NIBP hose should be completely dry before use, do not use wet NIBP hoses. These instructions were validated using the below detergents and disinfectants.

Cleaning agent (manual cleaning): Enzol® / Cidezime®, enzymatic detergent, Johnson&Johnson (add 8ml enzymatic detergent per 1 liter distilled water, for cable with dried-on organic matter use 16 ml enzymatic detergent per 1 liter distilled water/warm water)

Disinfectant (manual disinfection): Cidex OPA®, 0.55% ortho-phthalaldehyde solution, Johnson&Johnson (at a minimum temperature of 20°C/68°F).

Reverse osmosis/distilled water.

Caution: During the cleaning and disinfection of NIBP hoses, disposable gloves, protective eyewear, and a filtration mask should be used to reduce the risk of transmission of infectious agents by splashing or creating aerosols.

Warning: FMT NIBP hoses are sold NON-STERILE. Clean and disinfect the hoses before the first and every use.

Warning: Before cleaning or disinfecting the NIBP hoses, disconnect it from the cuff and the patient monitor.

Warning: NIBP hoses should be cleaned of overburden prior to disinfection to improve the effectiveness.

Warning: Do not sterilize NIBP hoses by autoclave, radiation, or steam.

Warning: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents.

Environmental Requirements:

Operating Temperature : 5°C - 45°C (41°F - 113°F)

Storage Temperature : 0°C - 50°C (32°F - 122°F)

Relative Humidity : 20 - 80% (not condensing)

The product should be protected from sunlight during storage. When not in use, hoses should be loosely coiled and stored at room temperature. It is recommended that the product be stored in its original package until the first use.

Compatibility: To ensure compatibility, the NIBP hose should only be used with the equipment for which they have been designed and is specified in the label on the product's primary packaging. For product models and compatible devices, please refer to the catalogue at metkomedical.com/PDF/05NIBPCuffsandHoses.pdf

Safety:

Degree of protection from electric shocks: type B

Degree of protection against the ingress of water: Keep dry

The FMT reusable NIBP hoses are sold NON-STERILE.

The FMT reusable NIBP hoses are not made with natural rubber latex and PVC.

The FMT reusable NIBP hoses have not been evaluated for safety in the MR environment. Scanning a patient who has this device may result in patient injury.

For an explanation of symbols, refer to the separate leaflet "Description of Symbols" located within the product package.

Reporting of serious incidents: Any serious incident related to the use of this product should be reported to both the manufacturer and the health authority/competent authority where the product is installed. Either contact your local representative or report to: metko@metkomedical.com

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Warranty: FMT reusable NIBP hoses are under twelve (12) months warranty against material and workmanship defects from the date of original purchase. In the warranty period, METKO will be responsible for repairing or changing the hose free of charge if the defect is proven. This warranty does not extend to any product that has been subject to misuse, wrong sterilization, neglect, or accident, or that has been damaged by causes external to the product, or that has been used in violation of the operating instructions supplied with the product. The product life is 3 years from the production date.

FMT® is a registered trademark of Metko A.Ş.

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